
# SAM XT – EXTREMITY TOURNIQUET


## INSTRUCTIONS FOR USE

Locate the site of injury\*\* and place the **SAM® XT** around the limb, directly onto the skin 2-3 inches (5-7 cm) above the wound.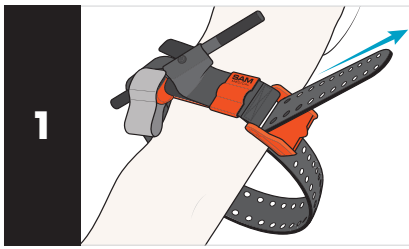


Insert the tip of the strap through the opening in the buckle. Grip the strap close to the buckle and pull hard and steady against the buckle until it stops. If prongs do not deploy, continue pulling until slack is removed. Maintaining tension, press the strap back upon itself to hold it in place.

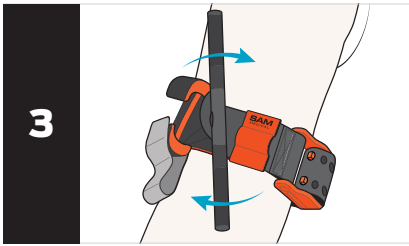Twist the windlass rod until the bleeding has stopped.

Fix the windlass rod inside the Windlass C-hooks and confirm that bleeding has stopped. If bleeding continues, try to further tighten the tourniquet with additional twists of the windlass rod. If this fails to stop the bleeding, apply a second **SAM® XT** side by side next to the first and ensure bleeding has stopped.

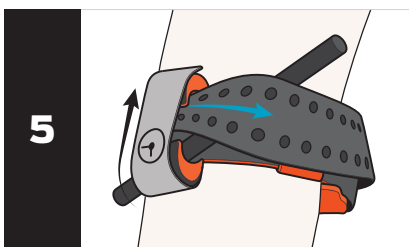Wrap strap between Windlass C-hooks, over rod and around the limb.

Secure the (⊕) Time Band over Windlass C-hooks and record time of application.

### ⚠ CAUTION ⚠

The **SAM® XT** **MUST NOT** be applied over solid objects within the clothing.

\*\* If the site of the injury is not easily seen, **place the SAM® XT as “high & tight” as possible on the injured arm or leg.**

Remove or reposition XT only under direction from a medical practitioner.

# SAM XT

### INTENDED USE:

The SAM® XT Extremity Tourniquet is intended to be applied around a limb to occlude arterial blood flow.

### INDICATION FOR USE:

To be used by first responders on patients with injured extremities within a circumferential range of 5.75 in – 32.5 in (14.6 cm - 82.5 cm) to control life threatening hemorrhage where direct pressure is not effective or is impractical.

### ⚠ PRECAUTIONS:

- The SAM XT must not be applied over solid objects.
- Leave tourniquet in place until directed by a medical practitioner.
- Dispose of used device per healthcare facility biohazard protocol.
- Device is single use only.
- Reuse of device may cause illness (e.g., infection) or injury (reduced performance).

### UNDESIRABLE SIDE EFFECTS:

- Tourniquet application may cause considerable pain.
- Application of the tourniquet may cause skin, muscle, vessel or nerve damage.
- Extended tourniquet application (>4 hours) may lead to rhabdomyolysis or the need for fasciotomy or amputation.

U.S. Patent No. 10,258,347 | 10,363,046  
SAM® is a registered trademark in the United States and other countries.

## REMOVAL INSTRUCTIONS

Leave tourniquet in place until directed by a medical practitioner.

### TO REMOVE SAM® XT

Release the (⊕) Time Band & untwist windlass rod.


Lift the strap to expose buckle. Then, lift the tab on the front of the buckle to allow prongs to retract.

If prongs do not retract from the buckle, cut the strap to remove the tourniquet.


### DISPOSAL

Dispose of device per facility policy.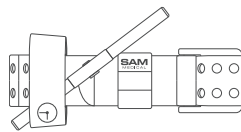

### MODELS

- **Tactical Black**  
XT600-BK-EN  
XT700-BK-EN\*
- **Hi-Viz Orange**  
XT600-OR-EN  
XT700-OR-EN\*
- **Hi-Viz Blue**  
XT600-BL-EN  
XT700-BL-EN\*

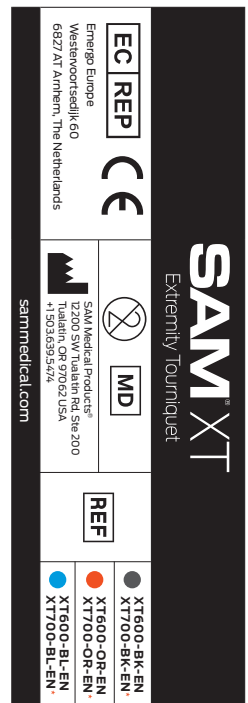©2024 SAM Medical Products®  
All rights reserved.  
350-0131-01-D ISSUE DATE: 2024-02

**Notice to Users and/or Patients:**  
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### SYMBOL GLOSSARY

Manufacturer	
Lot/Batch Code	LOT
Catalog Number	REF
Do Not Reuse	
Time Band Symbol	⊕
Caution	⚠
Medical Device	MD
Authorized Representative in the European Community	EC REP
CE Mark*	CE
Consult instructions for use, eIFU available at: sammedical.com/eIFU	
Importer	
Unique Device Identifier	UDI

\*May not be available in all countries

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