

QUIKLITTER™ LITE

REF #50-0469

Multi-Use Disposable Litter

Rated to 750 lbs (340 kg)

Size: 72" x 27"

Weight: 13 oz



RESCUE ESSENTIALS

www.rescue-essentials.com

info@rescue-essentials.com | 910-830-0286



Tri-Tech Forensics, Inc. dba Rescue Essentials
3811 International Blvd NE, Ste 100, Leland, NC 28451 USA



EC REP

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33,
Suite 123 2595AM The Hague
The Netherlands



LOT



01-0770R1

QuikLitter™ Lite Instructions

The QuikLitter™ Lite is rated to transport up to 750 pounds (340 kg). It is recommended to use at least four people to ensure safe transport and transfer, minimizing the chance of accident or injury. Not recommended when spinal stability is an issue. Improper handling or use of damaged QuikLitter™ Lite can result in death or serious injury. Please follow all instructions for best results.

CONTRAINDICATIONS AND WARNINGS

- Do not use QuikLitter™ Lite if unit is excessively worn or punctured.
- Avoid contact with sharp objects.
- Avoid dragging QuikLitter™ Lite over rough surfaces.
- Do not store QuikLitter™ Lite in contact with heat sources greater than 190°F.
- Not to be used in conjunction with any type of mechanical lifts for either vertical or horizontal movement.

PATIENT HANDLING

- Do not exceed load capacity of 750 pounds.
- Use a minimum of 4 people to lift patient.
- Ensure proper grasping technique before lifting patient.









CLEANING

- Soiled QuikLitter™ Lite can be cleaned using a damp cloth with soap, detergent, or a mild disinfectant.
- Do not machine wash.
- If the entire device is contaminated, place it in a red biohazard bag and dispose according to your hazardous waste protocol.
- Avoid use of harsh detergents or disinfectants.

INTENDED USE

The QuikLitter™ Lite is intended for use by healthcare professionals for general patient stabilization, transport, and transfer. Due to the lightweight and compact construction of the device, it is especially useful in austere environments where rapid casualty evacuations are needed.

Notice: Any serious incident that has occurred in relation to this device should be reported to Tri-Tech Forensics and the Competent Authority of the Member State in which the user and/or patient is established.

 MD	Medical Device	 EC REP	Authorized European Representative	 LOT	Lot Number	 PK	Country Code
 LATEX	Not Made with Natural Rubber Latex		Manufacturer		Consult Instructions For Use		Date of Manufacture

Made in Pakistan

RESCUE ESSENTIALS

www.rescue-essentials.com

910-830-0286 | info@rescue-essentials.com