

RESCUE ESSENTIALS

Mutual Aid Trauma Kit

Headrest Mount Installation Instructions



Raise headrest so there is several inches of clearance between headrest and seat.

Hold headrest mount to headrest with the strap D rings on the top of headrest as shown.



Pass vertical straps through space between headrest and seat.

Wrap horizontal strap around back of headrest and through D ring. Pull strap back on itself and secure with attached Velcro.



Wrap vertical straps up and around back of headrest and through D rings. Pull straps back onto themselves and secure with Velcro.



Place trauma kit into left side strap pocket then pull right side strap pocket over trauma kit. Secure with buckle.